



dermalogica® treatments with bt-sonic®

For a professional Dermalogica treatment with bt-sonic®, it is important to use Dermalogica products that are suitable for the client's current skin condition.

Step-by-step cleansing – with PreCleanse:

1. Dispense 1-2 pumps of **PreCleanse** into dry hands and apply to dry face. Massage to dissolve surface oil and dirt, concentrating on areas of congestion or stubborn debris. Wet hands to create a light milky emulsion.
2. Dampen the bt-sonic® brush head with water and apply to the skin. Gently work in circular motions around the face for about 1-2 minutes.
3. Rinse with warm water and sponges.

Step-by-step cleansing – with prescribed cleanser:

1. Select prescribed Dermalogica cleanser apply to a dampened face and throat, massage with damp fingers in light upward strokes.
2. Dampen the bt-sonic® brush head with water and apply to the skin. Gently work in circular motions around the face for about 1-2 minutes.
3. Rinse with warm water and sponges.

Step-by-step exfoliation:

1. Select level of exfoliation or a customized exfoliation according to your client's skin condition. Apply exfoliant as directed by Dermalogica®.
2. Dampen the bt-sonic® brush head with water and apply to the skin. Gently work in circular motions around the face for about 1-2 minutes, making sure not to over stimulate skin. Steam is optional.
3. Remove product as directed.

Sanitation with professional services:

Rinse with warm water and then run the device in a small bowl of sanitation product of choice following the manufacturers recommendation for time. Do not submerge the brush head. You may also spray the bt-sonic® cleansing head with a sanitation product of choice and then rinse.



dermalogica®